



# CareFlex

FLEXIBILITY IN A WORLD  
OF MIXED ABILITY

The Doge Collection User Guide  
**Modular Large/Magnus**



User  
Guide

## Dear Client

Your new Doge Collection chair has been designed to provide superior comfort and to be easy to use. The chair has many options so it can be adapted to meet your every seating need.

Please ensure that you read this concise user guide, especially the warnings and safety tips. We suggest that you keep this guide in the side pocket alongside the handset so that it is available for future reference.

### Tip for Operation

Because of safety requirements you cannot operate the chair's footrest when the chair is tilted forwards. You have to return the chair to the neutral position before you can extend the footrest.

### Maintenance and Cleaning

Regularly clean your chair with a vacuum cleaner and wipe with a clean damp cloth. If necessary dab stains with absorbent paper towels before cleaning with warm soapy water. **Do not use** aggressive cleaning agents and take care not to rub the fabric when cleaning. Always rinse with clear water. A detailed spot and stain removal guide is available on request.

### Handset Buttons



### Installation

- The Doge Modular is meant for indoor use, in a dry environment
- The Doge Modular requires a 220V / 240V power supply
- The working voltage of the Doge Modular transformer is 24V
- The power supply to the motors is switched off automatically when the handset is not in use
- Check that the power cable is not pulled too tight and does not create a tripping hazard
- If your chair is not going to be used for an extended period of time (e.g. during holidays) it is recommended that you unplug the transformer
- Ensure that there is sufficient space in front and behind the chair to extend the legrest and recline the backrest fully
- Do not position the chair too close to an open fireplace or heater
- Should the handset not operate, first check that the transformer is properly plugged into the socket. If it still does not work contact your supplier
- For any special maintenance or adjustment not mentioned in this user guide contact your supplier

### Safety tips

- Always grip the plug, not the cable, when disconnecting from the mains socket
- The Doge Modular is intended for single person use
- Never sit on the armrest or legrest
- When operating the handset ensure that there is nothing near the moving parts of the chair
- Do not allow children to play with the handset
- Never put anything under the chair
- Should an object get stuck in the chair or the base, remove the plug from the mains socket before attempting to remove it
- Take care to use little or no liquid when cleaning the floor on which the chair is standing
- If you accidentally spill liquid on your chair, first remove the plug from the mains socket. Then check whether the electrical components are dry. If not, carefully wipe them with an absorbent cloth and wait until completely dry before plugging back in
- Never open the housing of the electrical components

### Warning!

Do not attempt any repair work yourself – always contact your authorised Doge Collection supplier



Rehabilitation category winner - BHTA Excellence Awards 2008, 2009 & 2010

Approved code

CareFlex is a member of the British Healthcare Trades Association. For more information visit [www.bhta.com](http://www.bhta.com) or call 01732 458868

Customer Careline: 0800 0186440 [www.careflex.co.uk](http://www.careflex.co.uk) [enquiries@careflex.co.uk](mailto:enquiries@careflex.co.uk)



# CareFlex

FLEXIBILITY IN A WORLD  
OF MIXED ABILITY

## The Doge Collection User Guide Modular Large



Rehabilitation category winner  
BHTA Excellence Awards 2008, 2009 & 2010

[www.careflex.co.uk](http://www.careflex.co.uk)

For more information please contact:

- The Doge Modular meets the following EC directives:
- EMC Directive, 89/336/EEC, as amended
  - Low Voltage Directive, 73/23/EEC, as amended recently

CareFlex® and the wavy lines logo are all registered trademarks of CliniMed (Holdings) Ltd.