



# SmartSeat® Every Day Operation Quick Reference

## Seat Depth;




- 1) Bottom must be right back in the seat and a two finger gap between front of cushion and calves.
- 2) When the chair is empty and upright, squeeze the two bars together at the back of the chair and push/pull the back to the desired position. Let go of the bars and the back will click in to place.



## Seat Width;




- 1) Pull out tab at the back of the arm.
- 2) To make seat width wider, pull out the tab and push from the inside of the arm.
- 3) To make seat width narrower, pull out the tab and push from the outside of the arm.

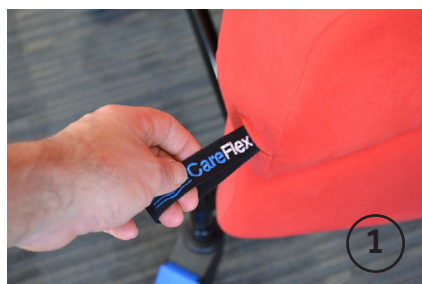

Release



increase seat width

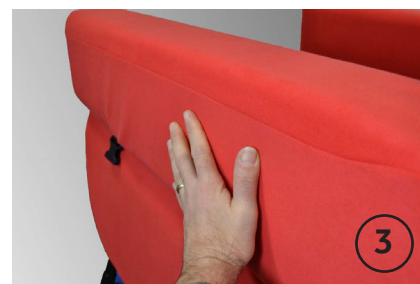

reduce seat width



## Legrest Elevation, manual chairs

Use the lever on the right side of the chair to release the legrest lifting mechanism. Assist the legrest to raise or lower to a comfortable position.



## Tilt in Space, manual chairs

Use the lever on the right push handle to release the Tilt in Space mechanism. Tilt the chair forwards or back to a comfortable position.



## Back Angle Recline, manual chairs

Use the lever on the left push handle to release the Back Angle Recline mechanism. Adjust the angle forwards or back to a comfortable position.

