




## LATERAL SUPPLEMENT

### FITMENT OF KIDNEY LATERALS

#### Position reference for Lateral Receiver Brackets

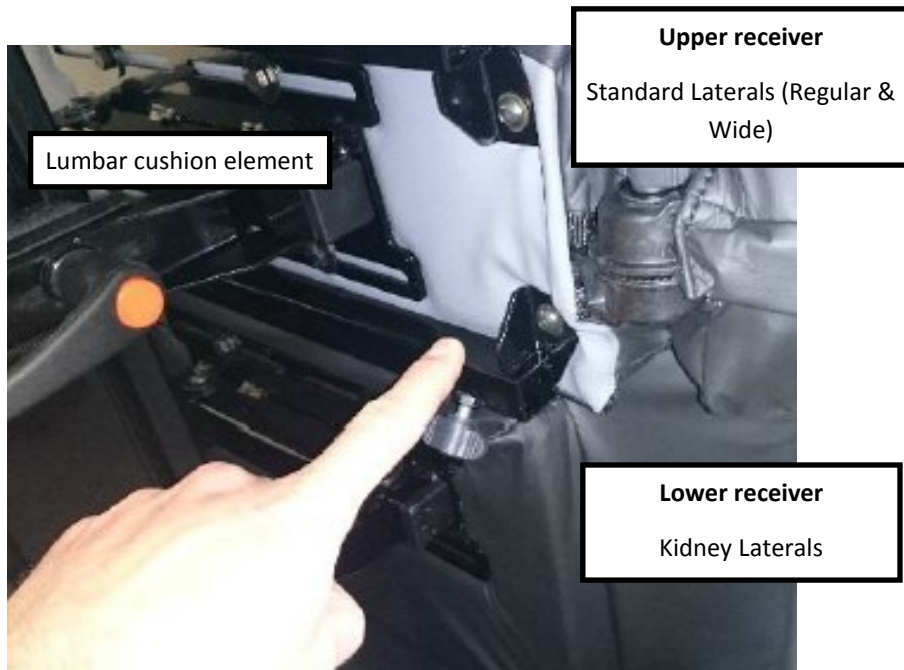

## LATERAL SUPPLEMENT

This supplement is applicable for the SmartSeat Pro Medium & Small, including the SmartSeat Pro Paediatric

**MUST BE READ IN CONJUNCTION WITH THE SMARTSEAT PRO USER INSTRUCTION BOOKLET**

The **SmartSeat Pro** can be fitted with adjustable laterals which have height and width adjustment to provide support to the sides of the rib cage. The adjustable laterals can be retrofitted to any **SmartSeat Pro** chair and each lateral can be independently adjusted.

### LATERAL OPTIONS & SIZES

The table below outlines the available options, ranges and codes:

Code	Description	Chest Width
T4LMR	Medium Standard, Regular	230-390mm
T4LMW	Medium Standard, Wide	300-460mm
T4LKR	Medium Kidney Lateral	275-395mm
ST4LMR	Small Standard, Regular	170-330mm
ST4LMW	Small Standard, Wide	240-400mm
ST4LKR	Small Kidney Lateral	210-290mm



## LATERAL SUPPLEMENT

### FITMENT OF STANDARD LATERALS (REGULAR & WIDE)


Each **SmartSeat Pro** is fitted with a single lateral receiving bracket as standard.

This is located on the rear of the lumbar cushion element and consists of a metal receiver tube.

When a customer orders a pair of Standard Laterals, they will also receive two plastic Handwheels that are used to secure the laterals into the metal receiver tube.

Simply slide each lateral into the metal receiver and tighten to secure in place

**Tip:** The wing on the lumbar cushion element can be rotated rearwards to make fitment of the accessory lateral easier

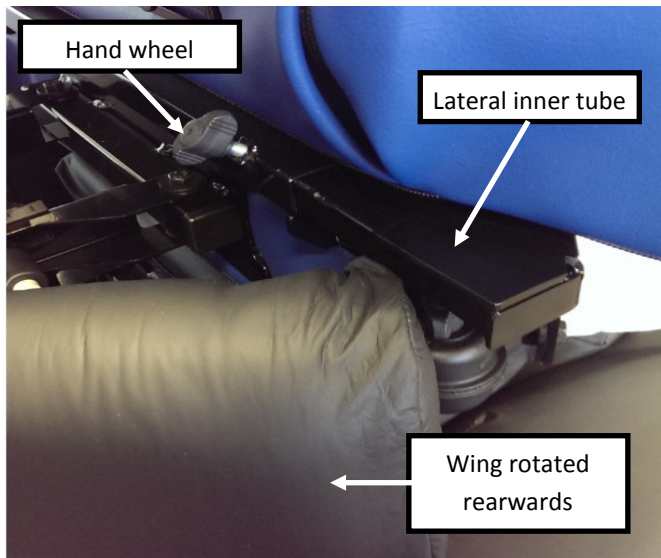

## LATERAL SUPPLEMENT

### FITMENT OF KIDNEY LATERALS

**Tools Required:** 4 mm allen key (not supplied)

An **additional** lateral receiving bracket must be fitted on the lower edge of the lumbar cushion element to enable Kidney laterals to be fitted onto the SmartSeat Pro. This additional bracket, plus fixings, will be supplied when Kidney laterals are ordered.

#### To fit;

1. Position the chair appropriately to permit easy access
2. The lateral receiving bracket for Kidney Laterals must be fitted on the **lower** edge of the lumbar cushion element, directly below the factory fitted receiving bracket
3. The lumbar cushion element is pre-prepared with fixings hidden underneath the upholstery. Using your finger, locate the depression in each corner of the upholstery—this signifies the position of fixing.
4. Use the Allen key to create a hole through the material—you may find there are already very small guide holes included in the material. Always check that these guide holes line up with the depressions felt before using the Allen key to increase the hole size.
5. Position and fix the additional lateral receiving bracket, using the fixings provided